

Extraction set for hip revisions


Universal system

freehand use without jig



Handle and extraction bolt


The extraction bolt can be attached on the top or bottom of the handle.

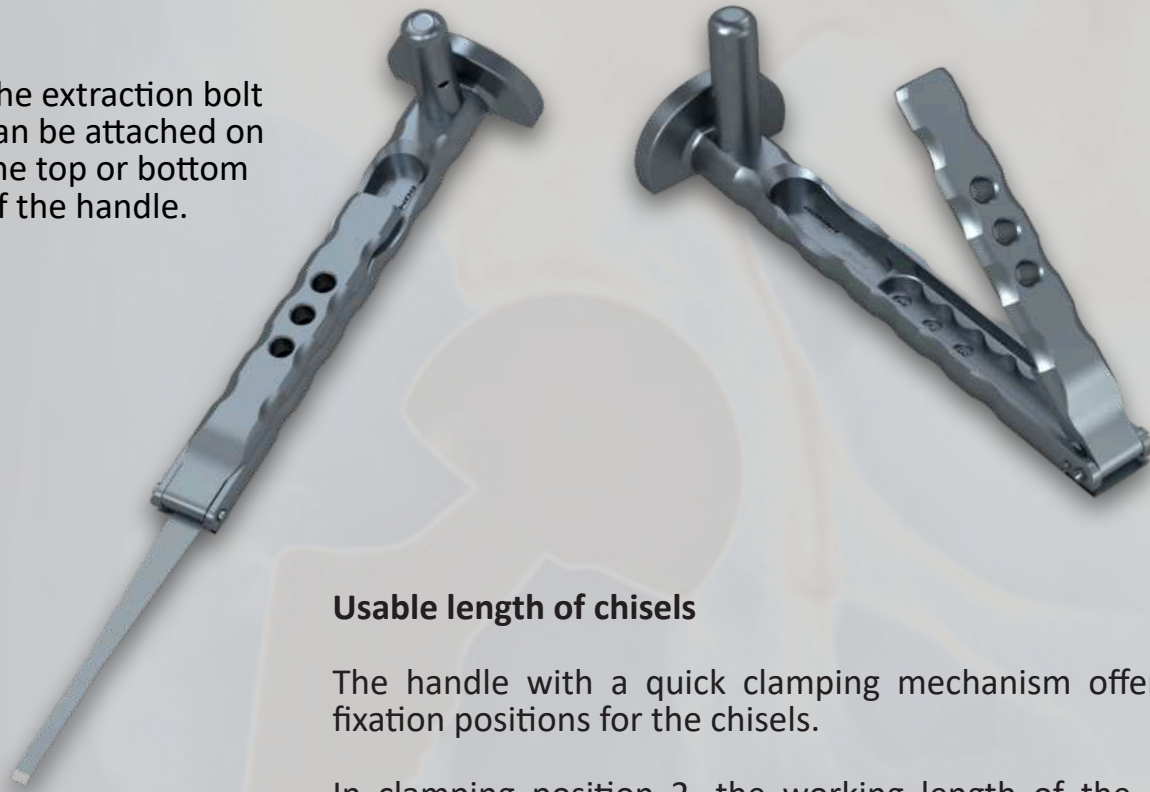

Usable length of chisels

The handle with a quick clamping mechanism offers three fixation positions for the chisels.

In clamping position 2, the working length of the chisel is extended by 15 mm compared to position 1, and by a further 15 mm in position 3. In total, the working length can be extended by 30 mm from position 1 to position 3.

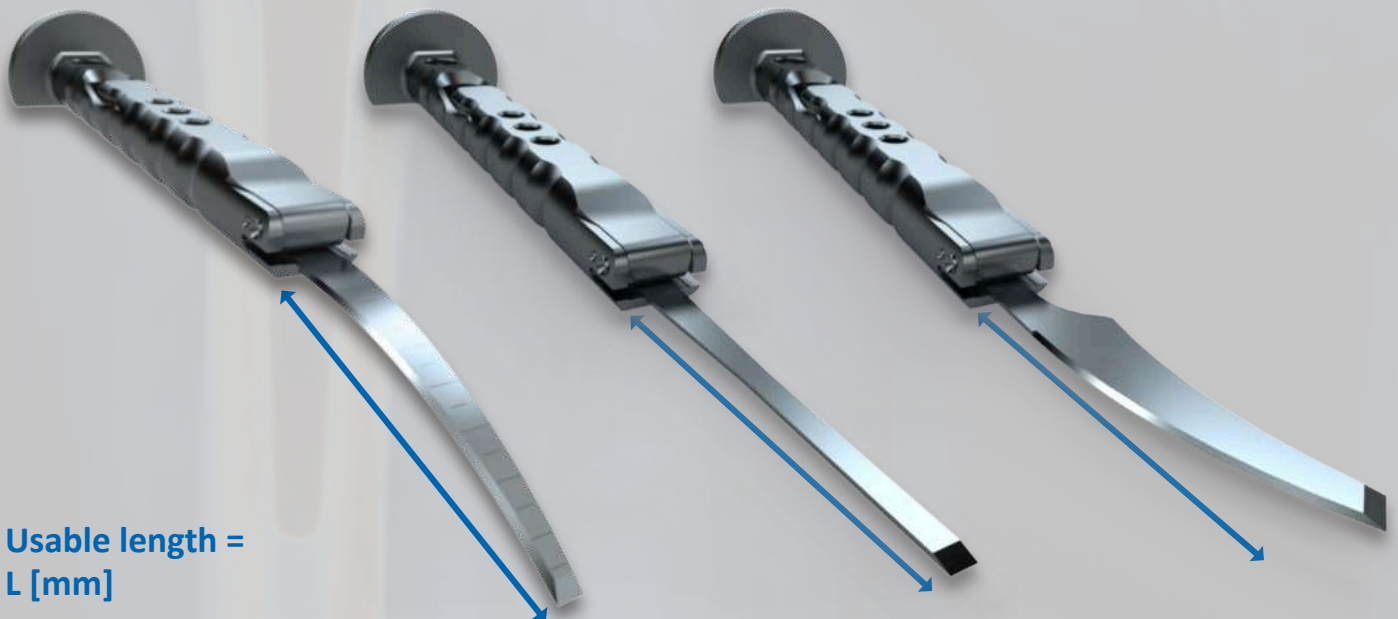

Usable length =
L [mm]

Standard universal chisels


The standard universal extraction set contains 6 chisels for free-hand use **without a jig-guided application**: 2 straight (short and long), 2 curved (medial and lateral) and 2 dorsal/ventral chisels. The design and dimensions of the chisels fit a large number of hip stems of the most common manufacturers. All chisels come in sterile packaging and are marked with the single-use symbol.

Reference	Article Description	L [mm]	W [mm]	T [mm]
401.101S	Chisel straight (long)	143.0	6.0	1.19
401.102S	Chisel straight (short)	83.0	6.0	1.19
401.201S	Chisel lateral curved	116.0	5.8	1.19
401.203S	Chisel medial curved	114.5	5.4	1.19
401.403S	Chisel ventral for right hip	118.4	9.5	1.19
401.404S	Chisel dorsal for right hip	118.4	9.5	1.19

Additional chisels



Reference	Article Description	L [mm]	W [mm]	T [mm]
401.401S	Chisel straight	100.0	5.0	1.19

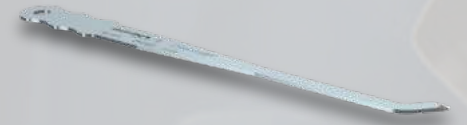

401.402S	Chisel straight	100.0	5.0	1.19
----------	-----------------	-------	-----	------



401.103S	Chisel straight	100.0	5.0	1.19
----------	-----------------	-------	-----	------




401.104S	Chisel straight	100.0	10.0	1.19
----------	-----------------	-------	------	------



401.105S	Chisel straight	100.0	15.0	1.19
----------	-----------------	-------	------	------



401.106S	Chisel straight	100.0	20.0	1.19
----------	-----------------	-------	------	------

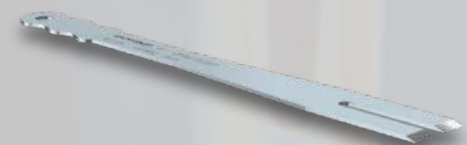

401.107S	Chisel straight	100.0	20.0	1.19
----------	-----------------	-------	------	------

Contact



Gomina AG
Raiftstrasse 4
3989 Niederwald, Schweiz
Phone +41 27 970 12 70
Fax +41 27 970 12 75
www.gomina.ch
sales@gomina.ch



GOMINA Deutschland GmbH
Ludwig-Vierling-Strasse 7
89233 Neu-Ulm, Deutschland
info@gomina.de

Version 2024-03
Last modification: 01.03.2024